

1. Description of the product

1.1 Product name: Surgical Gloves.


1.2 Intended use: Surgical Gloves is intended for use in surgery operation by doctors to help prevent contamination, cross-infection. It is for single-use and sterile unless the package is opened or damaged.

1.3 Materials: Natural latex.

1.4 Validity: 3 years.

2. Structure and parameters

2.1 Device structure

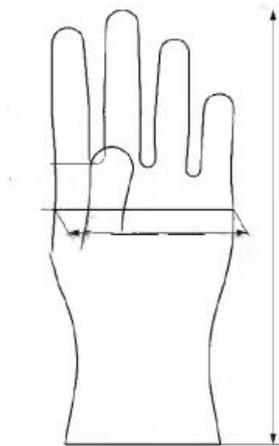

2.2 Device specifications

No.	Specification	Code	Width	Min Length	Min Thickness
1	6#	6	77±5	260	Smooth: 0.10 Textured: 0.13
2	6.5#	6.5	83±5	260	
3	7#	7	89±5	270	
4	7.5#	7.5	95±5	270	
5	8#	8	102±6	270	
6	8.5#	8.5	108±6	280	

According to powder, there are two types: pre-powdered, powder-free.



2.3 Product details

Material :	100% natural rubber latex	Sales packaging:	1pair/pouch ,100 pairs/box ,8 boxes/ctn
type	smooth and textured	Inner paper:	45*25cm
Testing insure:	AQL 1.5	Inner bag size:	16*14cm
MOQ:	100000pairs	Inner box size:	31*14*22.5cm
Country of origin:	Türkiye	Carton size:	32*61*46cm
H.S.code:	4015110000	Master carton CBM:	0.09 m3
Sterile ray	EO	G.W./ctn:	15,35kg

3.Device technical requirement

Property	Requirement
Water-tightness	No leakage shall be detected
Min. Force at break before accelerated aging /N	12.5
Min elongation at break before aging /%	700
Max force required to produce 300% elongation before accelerated aging /N	2.0
Min. Force at break after accelerated aging /N	9.5
Min elongation at break after aging /%	550

Powder Free gloves: Have a powder residual limit of 2.0 mg/piece.



4. Ontraindications, matters needing attention, warnings and Suggestions:

- 4.1. One-time use, no repeated use.
- 4.2. Do not use the product when its packaging is damaged.
- 4.3. The product shall not be used beyond the expiration date.
- 4.4. Adverse event history of the product: not found yet.
- 4.5. Suitable for people not sensitive to natural latex products. Patients with infectious diseases or skin ulcers in the hand cannot be used or carefully.

5. Usage and instructions:

- 5.1 is disposable and must be used immediately after opening the packaging and can not be used twice.
- 5.2 is prohibited if the package is damaged.
- 5.3 Please select the appropriate gloves before using the, to avoid breaking or shedding due to incorrect size.

6. Production date: see the packing mark.

7. Storage and transportation conditions:

- 7.1. The transportation requirements shall be specified in the order contract;
- 7.2. The products after packaging should be stored in a clean place at 10 to 25 degrees celsius
- 7.3. and the relative humidity should not exceed 80%.
- 7.4. There is no corrosive gas and well ventilated cleaning room.
- 7.5. the height of the stacking should not exceed 1.7M, no direct sunlight.

8. Expiration date:

- 8.1 after packaging shall be valid for 3 years from the date of production under the required storage conditions.